



it all starts with a smile...

### **Instructions for Val-Clean**

Val-clean Concentrated Denture Cleaner safely, quickly and effectively maintains the cleanliness of Valplast partials and other removable dental appliances. The solution may be used for a 15minute daily soak or for overnight soaking if staining develops.

### **Instructions**

- 1) Dissolve one spoonful (approx ¼ packet measuring spoon enclosed) into warm water in your denture container or small cup (approximately 250ml / 8.45fl.oz of water).
- 2) Allow solution to stand for 3 minutes.
- 3) Rinse the appliance before soaking.
- 4) Place the appliance in the solution for 15 minutes per day. (some appliances may need occasional overnight soaking to remove tough stains)
- 5) Rinse the appliance thoroughly under running water after soaking.

The solution may be used for 7 – 10 days before discarding. A very soft brush may be used periodically to remove loose particles. **Do not** brush with toothpaste or any other abrasive.

### **Caution**

We do not recommend soaking dentures containing special silicone soft – liners for more than 15 minutes per cleaning. This cleaner should **not** be used for appliances containing **Ticonium Alloy** as this may tarnish the appliance.

Powder and solution should never be used internally.

Swallowing the dust or solids may cause nausea or vomiting. **Do not** induce vomiting if ingested. If conscious, give water. **Get** immediate medical attention. **Do not** give anything by mouth to an unconscious or convulsing person. Keep out of reach of children and away from coloured fabrics.

Val-clean is a registered trademark of valplast supply services Inc.

Valplast is a registered trademark of valplast international corp.

Ticonium is a registered trademark of CMP industries Inc